

Tactical Articulating Splint

MANUAL



March 2023, ver. 01

Pub.No. 50-8101-020_01



Ferno Slovakia s.r.o.

913 07 Bosaca 893 Slovak Republic

Tel: + 421 (0) 32 7708 010, sales.sk@ferno.com



TIP: Regular cleaning helps reduce the risk of transmitting disease and enables the equipment to function at its optimum

Disclaimer

This manual contains general instructions for the use, operation and care of this Tactical Articulating Splint. Safe and proper use of this Tactical Articulating Splint is solely at the discretion of the user. Safety information is included as a service to the user. All other safety measures taken by the user should be within and under consideration of applicable regulations. It is recommended that training on the proper use of this Tactical Articulating Splint be provided before using this Tactical Articulating Splint in an actual situation. Retain this manual for future reference.

Tactical Articulating Splint Product Specification

Small size: 22,5 x 5,1 in
57 x 13 cm

Lightweight: 1500 g / 3,3 lb

| Product Number | Product Name |
|-----------------------|------------------------------|
| 60-0101-020 | Tactical Articulating Splint |



Tactical Articulating Splint Instructions

[Product Name] Medical Device: Tactical Articulating Splint [TAS]

- Safe and reliable equipment
- Easy use anywhere in the world
- Convenient use in various environments
- Designed for extreme conditions
- MOLLE system compatible



[Intended Use]

The Model Tactical Articulating Splint is used to immobilize and support injured lower and upper extremity.

- Simple concept
- Quick access system
- Comfortable handling

TAS can be used separately or in combination with Traction splint.



The TAS is marked with color marks corresponding with the color-coded straps. Each splint is equipped with all necessary lengths of securing color-coded straps for the lower and upper extremities.

RED and **GREEN** straps are used for legs

BLACK straps are used to buddy tie splint both legs

BLUE straps are used for arms



[Operating mode / Use]

The Model Tactical Articulating Splint in use

Place the upper edge of the TAS (marked HEAD) under the injured extremity as high as possible to the top of the Iliac Crest, so that the wider upper part of the TAS immobilizes the entire thigh and the narrow lower part of the TAS (marked FOOT) immobilizes the lower part of the extremity, ideally from the knee down.



Immobilize the lower extremity by fastening the red and green color-coded straps along the entire length of the lower extremity (as pictured).



Buddy tie splint both lower extremities with the black straps (as pictured).



Safe manipulation



[Extended Use]

Fracture and dislocation of the upper extremity

Immobilize the upper extremity by fastening the blue color-coded straps including a black strap that is buddy tied to the lower extremity.



[Product Features]

- Reusable
- Durable
- Waterproof
- Wisely equipped

[Cleaning and Disinfection]

The Tactical Articulating Splint is easy to clean and disinfect. Wipe all surfaces with disinfectant. Follow the disinfectant manufacturer's instructions for application method and contact time.

Ferno Slovakia Contact:


Ferno Slovakia s.r.o. Bošáca 893
913 07 Slovakia
Tel: + 421 (0) 32 7708 010

More information at: www.fernosk.sk



Warning

Please use under the guidance of the doctor or nurse

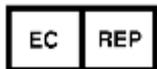

Ferno Washington Italia S.r.l.
via B.Zallone, n26, 40066
Pieve di Cento,
Bologna, Italy
+39.051.6860028

